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REF 8888268060, 8888268086

INSTRUCTIONS

- DO NOT close off blue air lumen while suction is applied to the tube.
- Check for firm connection/placement of the 5-in-1 adapter into the clear main lumen.
- If necessary, remove and insert firmly to prevent possible suction loss and leakage.
- After any irrigation procedure through the blue air lumen , reestablish an air buffer by injecting air with a syringe to clear the lumen of residual fluid.
- To cap tube, fit tapered end of blue vent lumen over 5-in-1 adapter.

RECOMMENDED SUCTION SETTINGS

- Always use the lowest suction setting that effectively decompresses the stomach.
- Continuous Suction: Set suction at a low level (30-40mm Hg (4-5.3 kPa)). If necessary, slightly increase suction until flow is observed.
- Intermittent Suction from a Central Source: Set suction at a low level, as with continuous suction.

REF 8888264911, 8888266106

INSTRUCTIONS

- (REF 8888266106 – Make sure blue end of Anti-Reflex Valve is firmly seated in blue air lumen vent.)
 - Seat 5-in-1 adapter snugly to prevent suction loss.
 - Follow any irrigation with an injection of air through the blue pigtail and then reseat the Anti-Reflex Valve into suction lumen.
 - (REF 8888266106 – To cap tube, fit white top of Anti-Reflex Valve into suction lumen.)
 - (REF 8888264911 – To cap tube, fit blue pigtail over 5-in-1 adapter.)
- #### RECOMMENDED SUCTION SETTINGS
- Always use the lowest suction setting that effectively decompresses the stomach.
 - Intermittent Suction: use 120mm Hg (16 kPa) max (e.g., Gomco, high setting).
 - Continuous Suction: use 30-40mm Hg (4-5.3 kPa) (low level). If necessary, slightly increase suction until flow is observed.
 - Intermittent Suction from a Central Source: Set suction at a low level, as with continuous suction.

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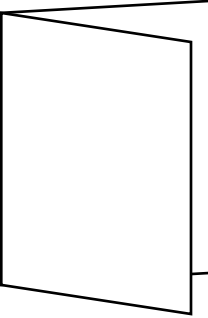
OPTION A			
JOB INFORMATION			
Product Brand and Descriptor:	Salem Sump™ Dual Lumen Stomach Tube		
REF #:	Generic		
GBU:	Medical Supplies		
REV:	06/2010		
ADDITIONAL JOB INFORMATION (if available)			
Job #:	58_KM	Dimensions:	17.00" W x 14.00" H
Artwork/Part #:	KM6420212	Metric Dimensions:	43.179 cm x 35.559cm
Print Process:	Preprinted	Package Type:	IFU
Barcode Format:		File Name:	KM6420212_GNRC_IFU
Check here if: <input type="checkbox"/> US Only <input checked="" type="checkbox"/> Generic Artwork			
Notes: Folded (8.5" x 14") (21.589 cm x 35.559 cm) TEMPLATE USED P54_IFU_8.5Wx11H_Flat_no_ill			

NUMBER OF COLORS					
					
PMS 281 C	XXXXX	XXXXX	XXXXX	XXXXX	XXXXX
TINTS					
					
XX%	XX%	XX%	XX%	XX%	XX%

TECHNICAL INFORMATION	
Application / Version:	InDesign / CS3
Placed Images:	None
Fonts Used:	Myriad Pro

COVIDIEN	
15 Hampshire Street Mansfield, MA 02048	
GBU-SPECIFIC INFORMATION	
Graphic Designer:	Lysa Amaral
DESIGN AGENCY INFORMATION (for external use only)	
Agency Name:	XXXXXX
Agency Contact:	XXXXXX
Job #:	XXXXXX
Log #:	XXXXXX

FOLDING PATTERN

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©2011 Covidien.